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
The small, easy to use bedside Mini Cradle offers flexible positioning of the pump at any angle or height.

Q Core's Mini Cradles are provided in one of the following configurations:


- a. With an optional connection splitter at the base (Figure 1, #3).
- b. With an optional Integrated Power Supply (IPS) at the base (Figure 2, #1). The IPS is an AC to DC power supply that is assembled to the Mini Cradle and is used to charge the pump battery.
- c. Without the optional connection splitter or the optional IPS at the base.

Identify your Mini Cradle configuration by turning the Mini Cradle bottom side up; Mini Cradles with a connection splitter will have two ports, as seen in Figure 1, #3. These are used for the RS-232 communication cable and power supply. Mini Cradles with IPS will have power cord connector covered by a power cord retainer, as seen in Figure 2, #2. This is used for Q Core's AC power cord.

*Figure 1*

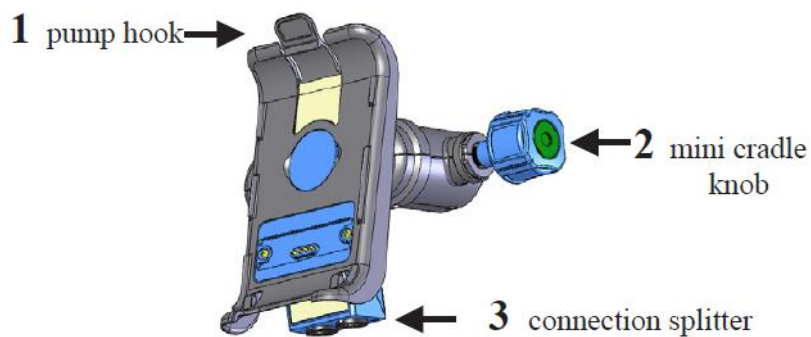*Figure 2*

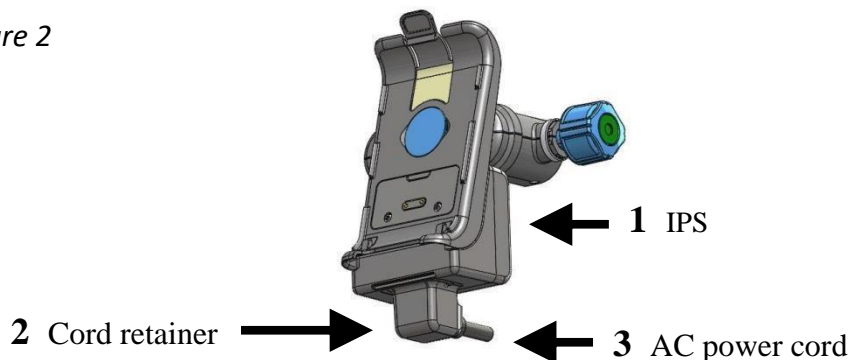**Directions for use:**

1. To attach the Mini Cradle to the IV pole: Place the open vice of the Mini Cradle on the IV pole and lock by turning the knob on the right (Figure 3, #2).


Ensure the cradle is securely attached to the IV pole before attaching the pump.

---

2. To attach the pump to the Mini Cradle: First insert the pump into the bottom hooks of the Mini Cradle (Figure 3, #1) and then click the pump into the top hook (Figure 3, #2).

Figure 3

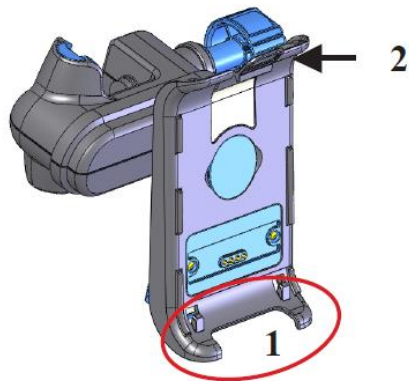3. To release the pump from the Mini Cradle: Hold the pump and press up the pump hook (Figure 3, #2).
4. To remove the Mini Cradle from the IV pole: Loosen and rotate the knob (Figure 1, #2).

To prevent theft – the Mini Cradle can be locked to the pole using the supplied plastic key. Removing the key from the knob will disable it and keep it in the locked position.


To attach several pumps to the IV pole, rotate the Mini Cradle to a horizontal position: Pull the pump holder away from the base of the Mini Cradle and rotate to the desired position.

---

### Connecting the AC power Cord to the IPS:

1. Plug the AC Power Cord connector (Figure 4, #1) to the IPS socket (Figure 4, #2).
2. Connect the cord retainer (Figure 4, #3) to the IPS by tightening the cord retainer screws with a Philips head screwdriver.
3. Plug the AC Power cord to the electrical outlet. Verify that the LED on the back of the IPS is on.

Figure 4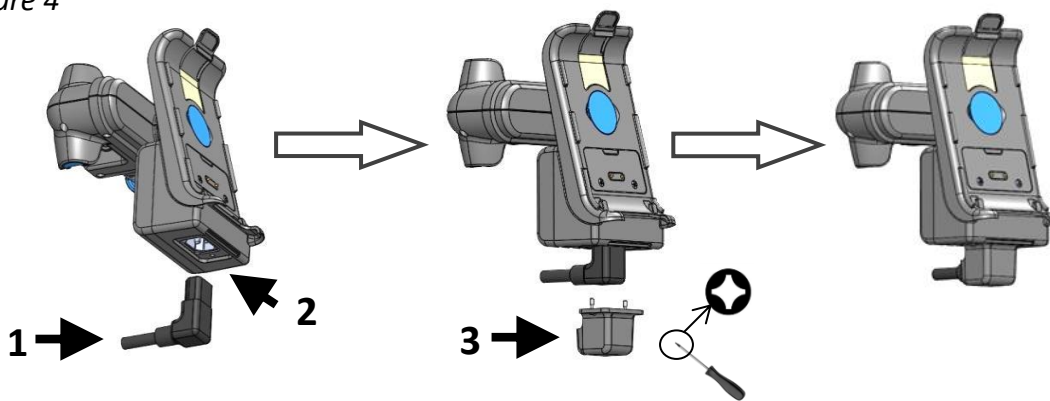


To be used with Q Core Medical's Mini Cradle and Q Core Medical's AC Power Cord only.

### Removing the AC Power Cord from the IPS

1. Disconnect the AC power cord from the electrical outlet.
2. Loosen the cord retainer screws with a Philips head screwdriver.
3. Disconnect the AC power cord from the IPS socket.

Figure 5

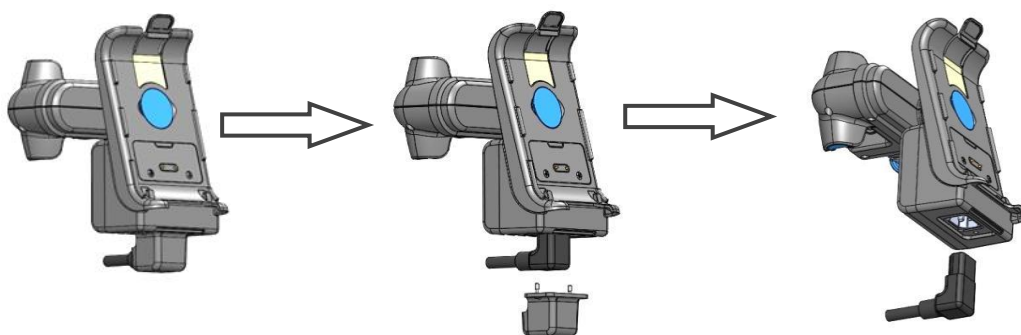## **Cleaning electric connectors Sapphire accessories**

Cleaning the electrical connectors of all accessories is restricted to use of 70% Isopropyl alcohol (IPA) ONLY

1. Place the accessory on a clean and stable surface
2. Apply IPA 70% lightly to a cloth or sponge
3. Squeeze the cloth / sponge before cleaning, so it won't drip on the accessory to be cleaned
4. Swab in a rotational manner for at least 3 bi-directional rotations (clockwise-counterclockwise)\*

5. Take caution and avoid dripping the reagent into the pins or pores of the electrical connector
6. Allow the IPA to air dry for at least 3 minutes before connecting to power

\*Be careful not to apply excessive pressure on the connector during swabbing

PCA 250 mL Lockbox

Mini Cradle with IPS

Splitter

Mini Cradle with Splitter

Sapphire External Battery Pack

Power supply Bolus handle / connector

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Mini-support avec connecteur de répartition ©